



## MAINTENANCE MANUAL

### **Cleaning suction inlet filter**

- Pull out filter part from machine in counter-clockwise direction of cover.
- Clean filter in running water for removing particle or debris.

(Recommending it cleaned at least twice per month for avoiding reduction of suction power)



- Re-connect filter

### **Cleaning suction unit**

**After every treatment:** For avoiding bad odor and reducing infection possibility, clinic should suck 1 glass of water before the next treatment

**After finishing last treatment of the day** – For avoiding bad odor, reducing infections possibility and assured a properly working of the unit, even to obtain extension of the warranty, dentist must clean suction holes sucking **NON-FOAMING** and **DISINFECTANT** liquids.



#### **CAUTION**

DO NOT use a foaming cleaning  
It causes backflow of water to the suction motor and can damage it

**After 1 year of working:** We kindly suggest, after 1 year of working, to change the aspiration filter and the non-return valve.